Patient Handbook

(Guide du patient)

Continuous Diffusion of Oxygen Therapy

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
Conseils utiles..... 26

Your EO₂ Device

In order to speed up the healing process, your EO₂ Device will deliver oxygen to your wound.

- **Wearable 24/7**
- **Portable**
- **Silent & Discreet**

The Flow Rate will be set by your healthcare provider.

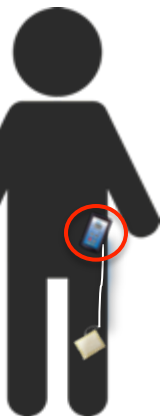Your Dressing

The OxySpur® Dressing is connected to the EO₂ Device to deliver the oxygen to your wound.

- **Super absorbent**
- **Comes off easy**
- **Gentle on skin**

Comes in
Adhesive &
Non Adhesive

Your Tubing

The Extension Set (tubing) carries the oxygen from the EO2 Device to the dressing.


- **Easy to connect**

Tip for Success:

Place tubing under clothing to avoid snagging it on objects.

Feed the tubing up the bottom of the pant leg and out through the waist band, and then connect it to the EO2 Device.

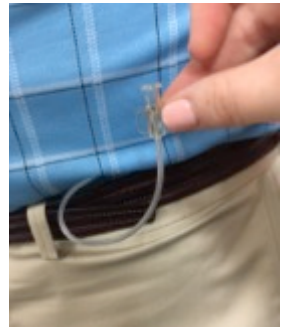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System Set Up


Connect your tubing to both the EO₂ Device and the OxySpur Dressing.

- 1) Connect the tubing to the EO₂ Device by twisting it onto the port marked "O₂" until snug.

- 2) Connect the other side of the tubing to the dressing by twisting it until snug.

DO NOT tighten the tubing too tightly.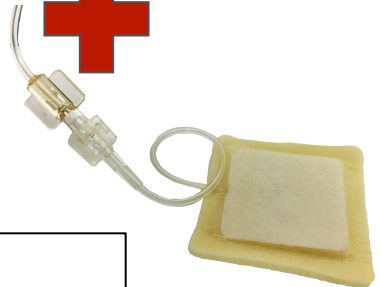

Dressing Changes

Change your dressing once a week unless instructed otherwise by your healthcare provider.


Applying the Dressing

1) Remove the dressing from the package.

2) **For adhesive dressings only**

Peel off the paper backing surrounding the dressing border, just like a BAND-AID®.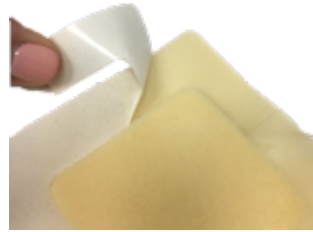

3) Place the dressing over the wound, foam side down, and secure it to the skin.

Tips:

- Your healthcare provider will provide you with details on how to secure your non-adhesive dressing.
- Don't worry about having an air tight seal on the dressing.
- Allow slack when securing the tubing to prevent it from pulling out of the dressing.

Charging

When the battery on your EO2 Device runs low, plug the device into the charger until it is fully charged.


***DO NOT turn your device off when charging.
Doing so will interrupt your therapy and slow down healing.***


- 1) Plug the charger into the “BAT” port on the top of the EO2 Device.


- 2) Plug the head of the power cord into the charger.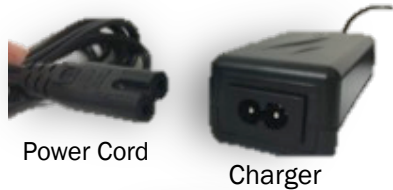

- 3) Plug the end of the power cord into an outlet.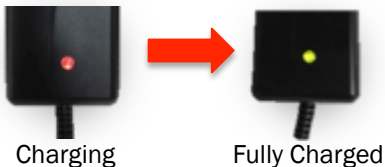

Unplug your device when it is **fully** charged:
When the red light on the charger changes from **red** to **green**.

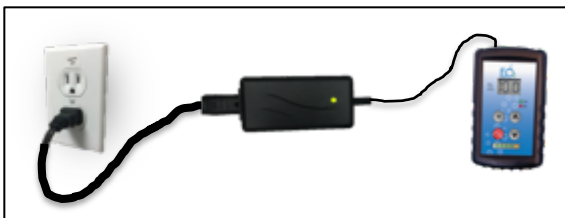TIP: Most patients find it easier to charge the EO2 Device overnight, like a cellphone.

Device Accessories

Carrying Case

Use the Carrying Case to make it easier to take your EO2 Device with you.

- 1) Hook the Strap onto the Carrying Case.

- 2) Unzip the Carrying Case.

- 3) Place your EO2 Device inside the Carrying Case face down and zip the case closed.

- 4) Turn over the Carrying Case and lift the Velcro top in order to see the device.

Device Accessories


Humidicant Pack

The EO₂ Device needs a humid environment.
The Humidicant Pack can be used to add moisture to your EO₂ Device, as needed.

Your healthcare provider will instruct you on when to use the Humidicant Pack.

- 1) Tear open and remove the outside package from the Humidicant Pack.

The inside pack should feel moist.

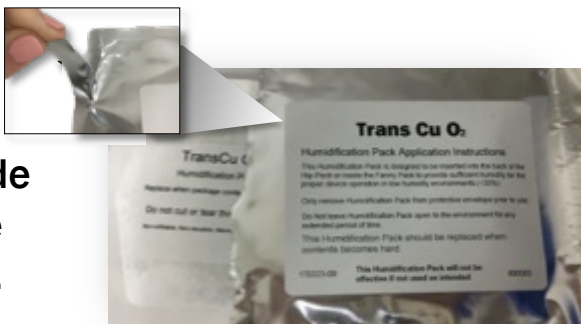- 2) Place the Humidicant Pack next to the device, inside the Carrying Case, and zip the case closed

Replace the Humidicant Pack when it is dry or feels “crunchy”

Alarms/Warnings

Beep for Low Battery

Your EO2 Device will beep when the battery is low.

Beep for Blockage

Your EO2 Device will beep if there is a blockage.
Press the MUTE Button to make the beep stop.

If an alarm continues, call your healthcare provider or:

1.800.825.2979

*For troubleshooting please consult the
Instructions For Use or go to eo2.com*

Tips for Success

Wear your device all the time (24/7)

DO NOT get the EO₂ Device wet

Only disconnect the EO₂ Device for bathing. When bathing, disconnect the tubing from the dressing and be sure to protect your dressing from getting wet.

Never turn your device off

Always leave your EO₂ Device on. Even when you disconnect it for bathing or charging.

DO NOT PEEK

Do not take off or adjust your dressing between dressing changes. Doing so will interrupt your therapy and slow down healing.

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